



## OrCam Read Audiostand



OrCam Read Audiostand - Manual.

Copyright © 2023 - Sensotec NV

All rights reserved.

Other product and company names mentioned herein may be trademarks or registered trademarks of their respective owners.

Version: 1.0.

Producer contact details :

Sensotec SA - Vlamingveld 8 - BE-8490 Jabbeke.

[info@sensotec.be](mailto:info@sensotec.be) - [www.sensotec.be](http://www.sensotec.be).

Head office :

Vlamingveld 8, B 8490 Jabbeke

T +32 50 39 49 49 - F +32 50 39 49 46

[sales@sensotec.be](mailto:sales@sensotec.be)

# Introducing the OrCam Read Audiostand

Thank you for choosing the OrCam Read as your solution to make you read the text aloud. Thanks to the special stand developed by Sensotec, blind users can also use this device as a reading device. The tripod is partly made of aluminium, measures 29 cm when unfolded and weighs only 475 g. Light and compact, you can therefore easily carry it in your bag. The ideal tool for working, reading, etc. at home and on the go, ensuring more independence in your daily life.

## Package contents

The kit you received contains the following items:

- The OrCam read holder combined with the Bluetooth speaker
- A tripod
- A screw
- Electrical adapter

Always check beforehand that all components are there!

## Care and maintenance

Clean your OrCam Read Audiostand with a damp, well-wrung cloth (NEVER use a wet blanket). Never use corrosive products: this will cause irreparable damage to the casing.

Keep the packaging in which the OrCam Read Audiostand was delivered. The device can only be shipped in this packaging.

Use only the mains adapter supplied. Use of an incorrect type may damage the unit.

Never switch on the unit if it has been exposed to water. Route all cables so that they cannot be tripped over or pinched.

## Warranty

Sensotec NV offers a warranty on the OrCam Read Audiostand for a maximum of two years after purchase, in accordance with the statutory warranty provisions.

Damage caused by inappropriate or negligent use is not covered by the warranty.

Maintenance and repair work may only be carried out by Sensotec nv technicians. The device must never be opened by the user or third parties. Should this happen, your warranty will expire without any obligation on our part.

## Technical support.

Finally, you can contact your local supplier for technical assistance. If the technical problem persists, or if you cannot find the answer to your question in this manual, please do not hesitate to contact us by e-mail ([support@sensotec.be](mailto:support@sensotec.be)) or by telephone on +32 50 40 47 47.

## The OrCam Read Audiostand components

### Overview

- The OrCam read holder combined with the Bluetooth speaker (a)
- A tripod (b)
- A screw (c)
- 5V power adapter (d)



# How to assemble the various components


- 1) Take the tripod (b) and assemble it with the the OrCam Read holder (a). You will notice that there is a raised dot (x) in the middle of the tripod and under the the OrCam read holder in the middle (x). This holds the two elements together.



- 2) Take the screw (c) and screw the two elements together.



- 3) Plug in the 5V power adapter (d).

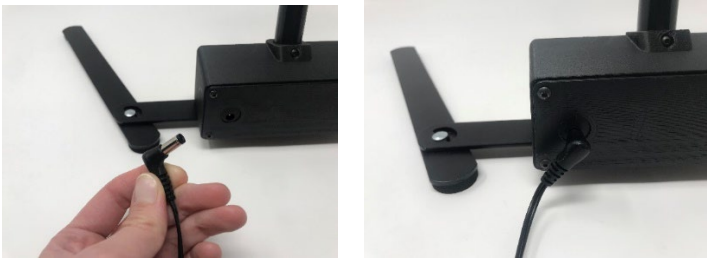

4) Turn on the switch.



## How to connect the device via Bluetooth :

1. Turn on the speaker. You will hear the start-up sound
2. Turn on the OrCam Read and open the audio settings menu via; Hold down the power button, press one of the volume buttons and then release the power button.
3. Go to the connect settings menu and then the Bluetooth settings menu.
4. Select "To connect to a Bluetooth audio device, press the trigger button; for other settings, press the volume buttons".
5. Select MH-M38 as the speaker and confirm.
6. You are now ready to use the speaker.

## Good to know about the OrCam Read Audiostand

If you activate the speaker and receive a notification intermittently, it is an indication that the battery is running low.

Charging with the supplied charger takes up to 3 hours.

An alternative method to enhance the battery life of the OrCam Read involves using the USB port located at the rear of the speaker to power the OrCam Read. This way it will charge the OrCam Read while using it.

The blue led on the front of the speaker shows the charge status of the speaker.





## Solutions for a visual challenge

Do you have technical or practical questions about this product? You can't find the answer in this manual? Then our Service & Support colleagues are here to help you!

### Service & Support

+32 50 40 47 47 | [support@sensotec.be](mailto:support@sensotec.be)

### Contact

[sales@sensotec.com](mailto:sales@sensotec.com) | [www.pro.sensotec.com](http://www.pro.sensotec.com)

**Sensotec HQ**  
Vlamingveld 8  
B-8490 Jabbeke  
+32 50 39 49 49

**Sensotec The Netherlands**  
Kalfjeslaan 38,  
NL-2623 AJ Delft  
+31 15 262 59 55