

Sensotec



Simplified Controller for LVI AIR schoolsystems

Simplified version of the LVI controller

Sensotec Simplified Controller for LVI AIR schoolsystems - Manual.

Copyright © 2024 - Sensotec NV

All rights reserved.

Other product and company names mentioned herein may be trademarks or registered trademarks of their respective owners.

Version: 1.0.

Producer contact details :

Sensotec SA - Vlamingveld 8 - BE-8490 Jabbeke.

info@sensotec.be - www.sensotec.be.

Head office :

Vlamingveld 8, B 8490 Jabbeke

T +32 50 39 49 49 - F +32 50 39 49 46

sales@sensotec.be

Introducing the Sensotec Simplified Controller for LVI AIR schoolsystems

Thank you for choosing the Sensotec Simplified Controller for LVI AIR schoolsystems as solution to use your LVI device. Thanks to this controller, specially developed by Sensotec, visually impaired students can easily manage the functions of their LVI device. The controller is made of..., measures ... and weighs only This is the ideal tool to give students more independence in their daily use of the LVI device.

Package contents

The product is made of

- A control panel with 9 buttons and a joystick
- A usb connection wire

Always check beforehand that all components are there!

Care and maintenance

Clean your Simplified Controller for LVI AIR schoolsystems with a damp, well-wrung cloth (NEVER use a wet blanket). Never use corrosive products: this will cause irreparable damage to the casing.

(Use only the mains adapter supplied. Use of an incorrect type may damage the unit.)

Never switch on the unit if it has been exposed to water. Route all cables so that they cannot be tripped over or pinched.

Warranty

Sensotec NV offers a warranty on the Simplified Controller for LVI AIR schoolsystems for a maximum of two years after purchase, in accordance with the statutory warranty provisions. Damage caused by inappropriate or negligent use is not covered by the warranty.

Maintenance and repair work may only be carried out by Sensotec nv technicians. The device must never be opened by the user or third parties. Should this happen, your warranty will expire without any obligation on our part.

Technical support.

Finally, you can contact your local supplier for technical assistance. If the technical problem persists, or if you cannot find the answer to your question in this manual, please do not hesitate to contact us by e-mail (support@sensotec.be) or by telephone on +32 50 40 47 47.

Simplified Controller for LVI AIR schoolsystems functions

Overview

This controller can replace the xbox controller without any modification to LVI's school set. Plug into a USB port and it's ready to use. Unlike the xbox controller, you do not need to turn it on.



What buttons and functions do we have ?

Bottom green button (camera symbol) : allows the user to select the camera that will be controlled by the controller. The selected camera is surrounded by an orange border on the screen. Of course, it is only possible to select another camera if several image sources are available.

Top green button : Used to set the split screen if there are multiple image sources.

Red button : converts the image from color to black and white and vice versa.

White button : allows the user to set high contrast.

Yellow buttons : allow to zoom in and zoom out.

Joystick : Use it to move the camera. It is possible to operate gradually. A small movement causes the camera to move slowly. If you choose the outer limits then the camera moves at its fastest.

Black buttons marked 1, 2 & 3 : these buttons match the preferred positions of the remote camera. Press these briefly to recall them.

Programming the preferred screen-positions

Programming the preferred screen-positions is done by pressing the joystick and pressing one of the black buttons simultaneously.

Setting brightness

Press the joystick and simultaneously press a zoom in or zoom out button to adjust the brightness of the image respectively.



Solutions for a visual challenge

Do you have technical or practical questions about this product? You can't find the answer in this manual? Then our Service & Support colleagues are here to help you!

Service & Support

sales@sensotec.com
BE +32(0) 50 39 49 49
NL +31(0) 15 262 59 55

Contact

sales@sensotec.com | www.pro.sensotec.com

Sensotec HQ
Vlamingveld 8
B-8490 Jabbeke
+32 50 39 49 49

Sensotec The Netherlands
Kalfjeslaan 38
NL-2623 AJ Delft
+31 15 262 59 55